


# Oticon ♦ Streamer

908 70 911 00 / 03.09




## Getting Started Guide

## Oticon Streamer Overview

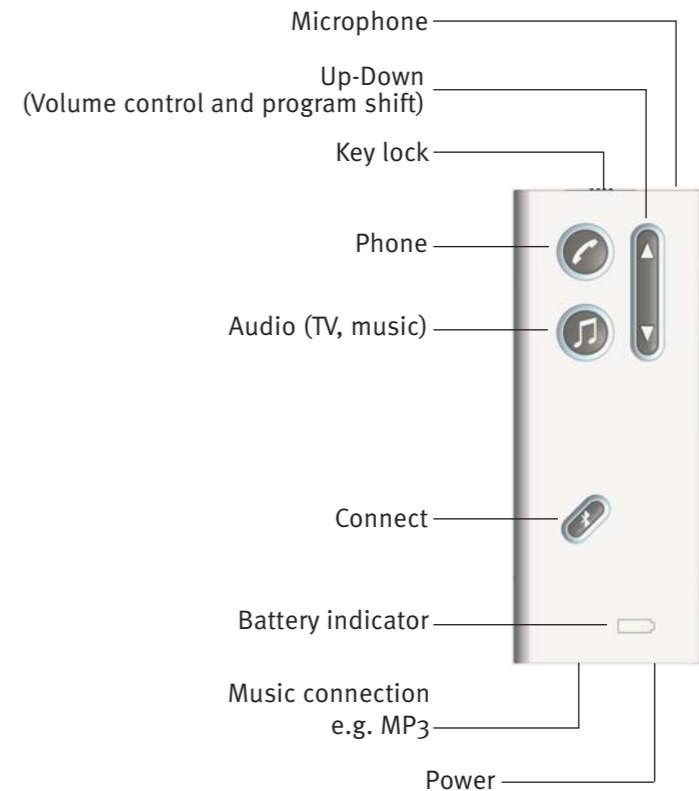

## Oticon Streamer Overview

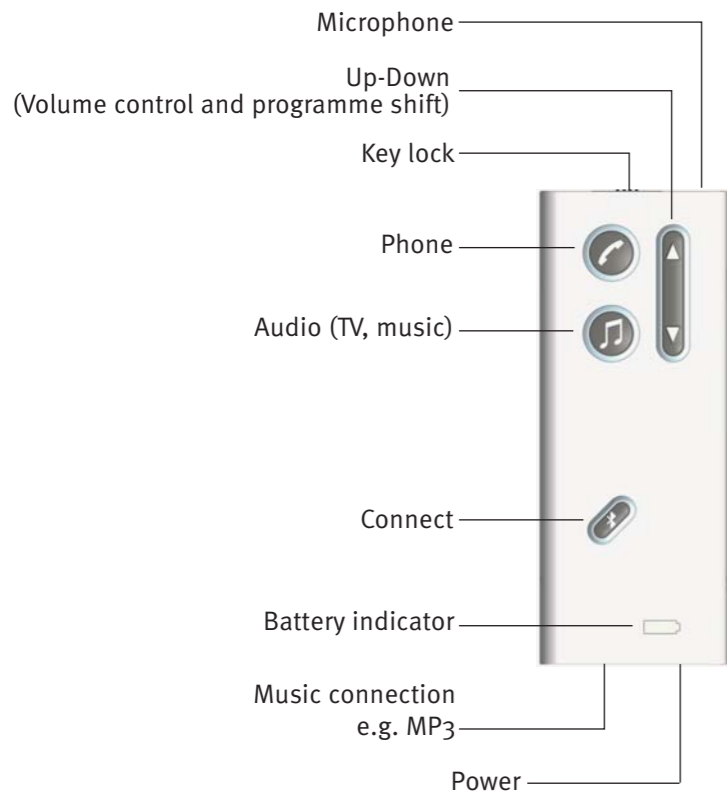

## Table of Contents

<b>Introduction</b>	3
<b>Wearing Streamer</b>	4
<b>How to get started</b>	6
<b>How to use with a phone</b>	7
<b>How to listen to audio (TV, music)</b>	8
<b>How to use as remote control</b>	9
<b>Important information</b>	10
<i>Battery</i>	10
<i>Key lock</i>	10
<i>On/Off button</i>	10

## My paired Bluetooth devices

---

---

---

---

---

---

---

---

---

---

## Introduction

**Streamer** connects your hearing instruments wirelessly to different audio sources and makes your hearing instruments work like wireless headphones.

Streamer transmits the sound directly into both hearing instruments and thereby improves the audio experience.

Streamer gives you the following possibilities:

- Hands free home phone conversations
- Hands free cell phone conversations
- Listening to TV sound
- Listening to sound (e.g. music) from a radio, PC, MP3 player etc. wired to Streamer
- Listening to sound (e.g. music) from a wireless stereo device via Bluetooth
- Remote Volume Control and Program Shift in the hearing instruments


## Wearing Streamer

To get the most out of your Streamer, wear it hands-free by using the included neck strap. There is a short and a long neck strap. If you use Streamer hand-held you risk occasional audio dropouts in the hearing instruments.

See the illustrations below on how to wear and use Streamer.

### Hands-free

*Neck strap*

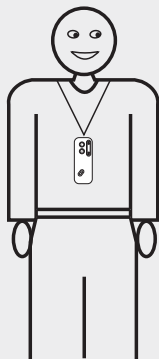

Purpose: Landline phone, cell phone, TV, music, remote control, stand-by


### Handheld

*In your hand*



Purpose: Landline phone, cell phone, remote control

*In your pocket*

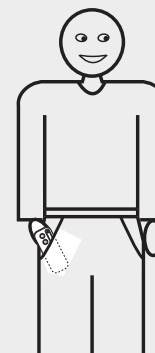



Purpose: Remote control and stand-by

## How to get started

### Pairing

Before Streamer can be used with your ConnectLine devices or a cell phone and music player, Streamer must be paired to these devices.

<b>Set Streamer into pairing mode</b>	Press and hold the Connect button for 5 seconds until the blue light flashes.  <i>After 2 seconds the blue light turns on, and after 5 seconds it flashes rapidly, which indicates the pairing mode.</i>	
<b>Turn on Bluetooth function</b>	The Connect function must be active to ensure the wireless communication between Streamer and your audio sources.  Press the Connect button shortly. The blue light turns on.  After a few seconds the blue light starts to pulsate and after half a minute it turns off again to save power.	


*Refer to Streamer manual for detailed pairing instructions.*

For more information on pairing your cell phone with Streamer, visit the Oticon website on [www.oticon.com](http://www.oticon.com)



## How to use with a phone

### When talking


Talk into the microphone of Streamer (optimal up to 30 cm/12 inch distance).

<b>Answer incoming phone calls</b>	When a phone connected to Streamer rings, you will see a blinking green light around the Phone button and you will also hear a ringing tone in your hearing instruments.  Press the Phone button shortly to answer the call. The ring around the Phone button now lights solid green.  <i>(The Connect function must be active – see “Turn on Connect function”)</i>	
<b>Outgoing calls with ConnectLine Phone</b>	Press Streamer’s Phone button. The hook is off and you will hear a dial tone in the hearing instruments. The green rim light shows constant light in Streamer’s Phone button. Pick up the phone receiver and dial the wished phone number. Put the receiver back on.	
<b>Outgoing calls with your mobile phone</b>	Use your mobile phone to make the call. When the call goes through, talk into Streamer – do not press the Phone button on Streamer.	

## How to listen to audio (TV, music)

<b>Listening to ConnectLineTV</b>	Press the Audio button when near the TV. After a few seconds the orange shows constant light and the hearing instruments receive theTV sound.	
<b>Listening to wired music</b>	Connect the cable of the audio source to the Music Connection of Streamer. Press the Music/Audio button. The orange ring around the button lights up and the music starts.	
<b>Stop TV listening or music</b>	Press the Audio button. The orange light in the button turns off.	
<b>Adjust volume</b>	Use the Up-Down button. Press Up or Down until the volume in the hearing instruments is comfortable.	

## How to use as remote control



<b>Adjust volume</b>	Press Up or Down shortly to adjust volume. Repeat until the volume in the hearing instruments has reached the desired level.	
<b>Shift programme</b>	Press and hold Up or Down for 2 seconds to go forwards or backwards in the programme cycle. <i>Only possible if more programmes are present in the hearing instrument.</i>	

# Important information

## Battery

The Streamer has a built-in rechargeable battery. The battery will last 5 hours when audio streaming is active.

When Connect function is on (Stand-by) the battery will last for 70 hours.

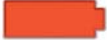The battery indicator will light up solid red when the battery is low (about 20 minutes left). The battery indicator will start blinking when the battery is close to empty (about 5 minutes left).	
Recharging the battery takes approximately 4 hours. When charging the battery, the indicator blinks green. When fully recharged, the battery indicator lights solid green.	

The battery should be charged frequently.

## Key lock

You can lock the keys to avoid pressing the buttons unintentionally. The key lock slider on the top of Streamer locks and unlocks the keys.

## On/Off button

The Streamer has no On/Off button, as it only uses little power. If you plan not to use Streamer for longer periods, turn off the Bluetooth function.

The slider button on top of Streamer is a keyboard lock, not an On/Off switch.