

STREAMER FIRMWARE RELEASE HISTORY

Streamer version	ConnectLine				
	1.0	1.1	1.2	1.3	1.4
Firmware	2.1.4	3.0.7	3.7.8	3.8.4	
Main features	HI Volume up/down	HI Volume up/down	HI Volume up/down	HI Volume up/down	HI Volume up/down
		Program Shift HI	Program shift long press Vol up/down	Program shift long press Volume up/down	Program shift long press Volume up/down
		Mute HI long press Audio (when streaming)	HI mute long press Vol up/down	HI mute long press Vol up/down	HI mics mute long press Vol up/down
			Connect light stops fading	Connect light only stops fading when key-lock	Connect light only stops fading when key-lock
	1 Audio source, 1 Phone and minijack	1 Audio source, 1 Phone and minijack	MultiConnect AutoConnect AutoPriority AutoResume	MultiConnect AutoConnect Auto Priority AutoResume	MultiConnect AutoConnect AutoPriority AutoResume
				Transfer call to cell phone (long press Phone Button when connected to cell phone)	Transfer call to cell phone (long press Phone Button when connected to cell phone)
				Secondary A2DP can be started with long push Audio Button	Secondary A2DP can be started with long push Audio Button
(HI=Hearing Instrument)					ConnectLine Microphone can be started with double press Audio Button*
Other:					
ConnectLine compatibility					
ConnectLine TV Adapter			•	•	•
ConnectLine Phone Adapter			•	•	•
ConnectLine Microphone*					•*
Misc. Streamer					
Firmware version (Key lock on, Vol up and Vol down. Wait 10 secs. FW 1.4 = 5 blinks)					
Reset Streamer (Key lock on, long press phone button) LEDs flash once					
Clear Pairings (Key lock off, long press Connect Button and Volume up) Backlight turns off					

* New and/or changed with latest release